MobiloFix[®] Epi

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Elbow brace

Elleboogbandage

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Dear customer,

We thank you for choosing our MobiloFix® brace. MobiloFix® braces are manufactured in accordance with the state of the art. We use best materials and deliver products of extraordinary quality. A stable quality management assures our customers to find braces that can withstand any requirements: stabilizing, constant compression, breathable materials with a pleasant feeling to be worn, as well as good fitting.

This brochure explains thoroughly how to use and take care of your brace. Please read the hints before using the brace for the first time. Please ask your practitioner or a medical professional near you if any questions appear.

Intended Use

MobiloFix® Epi both stabilises and relieves the elbow joint when it has been subjected, for example, to stress related and other tendon type inflammation – also known as „tennis arm“ and „golfer arm“.

Warnings

- Too tight bandaging can have a detrimental effect and should therefore be avoided. Further injuries can also occur as a result of incorrect selection of the bandage size or by the incidence of unfavourable joint posture (intense bending of the joint) and/or if the bandage has been worn for too long a time.
- The bandage should be worn during movement.
- Please consult your doctor if swelling occurs, or if there are changes in the appearance of

the skin or if there are any other indications connected with the wearing of the bandage.

Contra-indications

The bandage is not to be used or may only be used subject to your doctor's agreement in the following cases:

- Skin, muscle, tendon and bone injuries in the vicinity of the bandage,
- Massive, local soft tissue and vessel inflammation in the vicinity of the bandage,
- Advanced circulation disorders in the vicinity of the bandage and/or extremities,

- Thrombosis afflicts in the vicinity of the bandage and/or extremities,
- Massive soft tissue swelling and haemorrhages in the vicinity of the bandage,
- Advanced neuropathy in the vicinity of the bandage,
- Where incompatibility with the bandage material occurs.

How to attach and fasten your brace

For best use your brace must be attached correctly. The size is also very important. Braces that are too small can be too tight,

compromise the blood circulation, lead to a feeling of uncomfortable skin irritations and unwanted pressure. Braces that are too big cannot hold correctly, therefore they do not fulfil their function, and often drift out of their defined place.

You shall, therefore, always assure that the brace is fitting perfectly. On the back side of this brochure you will find a size table and hints to find the right fitting parameters and dimensions. You can also read the size on the brace itself. Above the logo you will see several bars.

The number of bars represents the brace size. Take the brace into your hand so that the two logos are on top. The seam shall point to the back. Below the logos you feel supporting rings inside the material (fig. 1). Put on the brace on your arm so that the rings get to lay on your elbow joint. The seam shall be exactly in the back and lie across the middle of the elbow (fig. 2). Take care that the brace lies flat and does not roll up at the ends (fig. 3). The brace shall feel tight without sliding out of place or compromising your arm (fig. 4).

Pay attention to this

MobiloFix® Epi was exclusively manufactured for use on the elbow joint. Please do not use the brace on another body part. Please note that MobiloFix® Epi can be used on either right or left elbow. MobiloFix® Epi stabilizes and disburdens the joint, for instance after injuries, at overload affects and tendinitis. The brace must not be used for rehabilitation of fresh fractures or open wounds. Please let the brace be prescribed by a practitioner and follow his using advice.

How to take care of your new brace

If you take good care of your new brace, you shall benefit from its use for a long time. Please avoid the contact with balms, crèmes or other material to the brace. Wash your brace with a mild detergent at 30°C hand washing. Do not wring out the brace after washing! It is better to press the brace a little into a towel and to let it naturally dry at the air in a not too warm, not too sunny place. In this way the brace's shape and flexibility remains.

Attention: Read accompanying documents, especially user manual

By labelling with CE certificate, the manufacturer states that the product meets all active requirements of the regarding EU Directive. A conformation evaluation process has been successfully completed.

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Pierenkemper GmbH