MobiloFix® Lumbo

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Back brace

Rugbandage

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Dear customer,

We thank you for choosing our MobiloFix® brace. MobiloFix® braces are manufactured in accordance with the state of the art. We use best materials and deliver products of extraordinary quality. A stable quality management assures our customers to find braces that can withstand any requirements: stabilizing, constant compression, breathable materials with a pleasant feeling to be worn, as well as good fitting.

This brochure explains thoroughly how to use and take care of your brace. Please read the hints before using the brace for the first time. Please ask your practitioner or a medical professional near you if any questions appear.

Intended use

MobiloFix® Lumbo was exclusively manufactured for use on the lower back. Please do not use the brace on another body part. MobiloFix® Lumbo both stabilises and relieves pain in the low back.

Warnings

- Too tight bandaging can have a detrimental effect and should therefore be avoided. Further injuries can also occur as a result of incorrect selection of the bandage size or by the incidence of unfavourable joint posture (intense bending of the joint) and/or if the bandage has been worn for too long a time.
- The bandage should be worn during movement.
- Please consult your doctor if swelling occurs, or if there are changes in the appearance of

the skin or if there are any other indications connected with the wearing of the bandage.

Contra-indications

The bandage is not to be used or may only be used subject to your doctor's agreement in the following cases:

- Skin, muscle, tendon and bone injuries in the vicinity of the bandage,
- Massive, local soft tissue and vessel inflammation in the vicinity of the bandage,
- Advanced circulation disorders in the vicinity of the bandage and/or extremities,

- Thrombosis afflicts in the vicinity of the bandage and/or extremities,
- Massive soft tissue swelling and haemorrhages in the vicinity of the bandage,
- Advanced neuropathy in the vicinity of the bandage,
- Where incompatibility with the bandage material occurs

How to attach and fasten your brace

For best use your brace must be attached correctly. The size is also very important. Braces that are too small can be too tight, compro-

mise the blood circulation, lead to a feeling of uncomfortable skin irritations and unwanted pressure. Braces that are too big cannot hold correctly, therefore they do not fulfil their function, and often drift out of their defined place.

You shall, therefore, always assure that the brace is fitting perfectly. On the back side of this brochure you will find a size table and hints to find the right fitting parameters and dimensions. You can also read the size on the brace itself. Above the logo you will see several bars. The number of bars represents the brace size.

Please stand while attaching the brace. Make sure that the correct side is up. You can recognize it by the bars that represent the size, as they must point upwards. The ends of the brace, moreover, point upwards in a light curve. Hold the brace behind your back so that the stabilizing stripes point to the body. Hold the ends of the brace tight so that right and left have the same length. In order to do so you may let your fingers through the loops (fig. 3). After this, bring your hands in the height of the lower back to the front to fasten it (fig. 4) with the Velcro strip. Pull on the brace until it

fits properly (fig. 5). The brace shall feel tight without sliding out of place or compromising your back or breathing (fig. 6). The lower back shall be well-supported.

The same is true for the application of a so called pelotte (fig. 1). Please position the pelotte with the Velcro stripes onto the regarding areas in the centre of the bandage (fig. 2). The pelotte (closed V shape) points downwards with the slim side, the broad side is above. The pelotte suits correctly when positioned in a central way on the spine.

Instructions for use MobiloFix® Lumbo

Pay attention to this

MobiloFix® Lumbo stabilizes and disburdens the lower back, and therefore it comforts the back in case of pain. The brace must not be used for rehabilitation of serious back injuries or acute herniated discs. Please let the brace be prescribed by a practitioner and follow his using advice.

How to take care of your new brace

If you take good care of your new brace, you shall benefit from its use for a long time. Please avoid the contact with balms, crèmes or other materials to the brace. Wash your brace with a mild detergent at 30°C hand washing. Do not wring out the brace after washing! It is better to press the brace a little into a towel and to let it naturally dry at the air in a not too warm, not too sunny place. In this way the brace's shape and flexibility remains.

Attention: Read accompanying documents, especially user manual

By labelling with CE certificate, the manufacturer states that the product meets all active requirements of the regarding EU Directive. A conformation evaluation process has been successfully completed.

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Pierenkemper GmbH