DE / EN

Gebrauchsanweisung Art.-Nr. 101677-V02

**schwa-medico**  
MENSCHLICHE MEDIZIN

# PierenHeat TDP / PierenHeat<sup>2</sup> TDP

## INFRAROT-WÄRMELAMPE

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**INFRARED HEATING LAMP**

# ABBREVIATED INSTRUCTIONS

- » Use the telescopic stand to set the required height. The flexible arm allows the heating head to be brought into the required position.

- » Adjust the heating head in such a way that by sitting comfortably, the rays from the lamp are directed to the required part of the body.

- » Connect the heating lamp to the mains power supply.

# ABBREVIATED INSTRUCTIONS

4 » Switch on the lamp with the Power button. The pre-heating phase is started by depressing the button Start/Pause on the control panel. The green LED illuminates. On the display there appears "cont". The lamp does not switch off automatically in this mode.

5 » Depress the Reset button on the control panel. Finally depress the timer button to set the treatment time and start treatment with the Start/Pause button. Please note that the pre-heat time lasts approximately 10 - 15 min.

6 » Please ensure that during treatment the lamp head continues to be directed towards the required part of the body. On no account should you, during the treatment time, touch the warming head. Once the set treatment time has elapsed, the heating lamp does switch off automatically in this mode.

## Content

Important Instruction – please read before using the lamp	21
Purpose Of Use	21
Indications	21
Function	21
Safety instructions	22
Contra-Indications	23
Side-Effects	24
Function Of The Heating Lamp: How it should be used	24
Operating elements	24
Operating panel	25
Preparations for treatment	26
Assembling the heating lamp	26
Realisation of the treatment	27
Pre-heating	27
Treatment	27
Recommended treatment duration	28
After treatment	28
Replacing the heating head	28
Heating head disposal	29
Technical Data And General Information	29
Care and cleaning	29
Classification	29
Delivery content PierenHeat TDP	29
Delivery content PierenHeat <sup>2</sup> TDP	29
Accessories	29
Warranty	29
Technical data	30
Picture symbols	31

***Dear Client,***

We are very pleased that you have decided to purchase this heating lamp. These operating instructions explain the device and its application. In addition, these instructions contain numerous important notes and suggestions regarding treatment. You are therefore requested to read these instructions carefully before use.

**Important Instruction – please read before using the lamp*****Purpose Of Use***

The heating lamp is a medical product which has been designed to warm the skin and underlying tissue by means of infra-red radiation. The lamp should be used for no other purpose and observance of the safety instructions in these operating instructions is necessary at all times.

***Indications***

Certain locomotor system complaints can be alleviated with the help of the heating lamp.

These include:

- » Complaints in the area of joints caused by arthrosis, such as pain and stiffness in the joints.
- » Complaints in the area of muscles and tendons such as pain, sclerosis and tension.

***Function***

The heating lamp emits infra-red light waves which penetrate and warms the skin; this has a number of effects:

- » Stimulation of blood circulation
- » Alleviation of muscular cramps
- » Reduction of stiffness in the joints
- » Pain relief
- » Improved tissue elasticity

The heating lamp is to be used as a treatment support.

**Caution:** Health disorders and pains can be a sign of serious illness and even where the use of the heating lamp has positive effects it should not therefore be assumed that the underlying cause of the pain has necessarily been cured. Please consult your doctor.

### ***Safety instructions***

Please read these instructions on safety carefully before using the lamp.

- » Before connecting the heating lamp to the mains power supply please check compliance with the voltage data which is stated on the rear side of the control panel.
- » Avoid over-heating the heating lamp. It should always be secured on a stable, even surface. Do not attach the heating lamp to walls or hang from the ceiling. Neither should it be placed on uneven or sloping floors.
- » Ensure that the power cable is clearly visible and does not represent any sort of obstacle.
- » Do not clean the heating head with liquids.
- » Avoid strong magnetic fields in the vicinity of the heating lamp, for example those emanating from a mobile telephone.
- » Never cover the lamp, for example with material or clothes when it is in operation.
- » Keep the lamp well away from water or other liquids.
- » Do not use the heating lamp in a bathroom or similar rooms.
- » Do not expose the heating lamp to vibration or shaking.
- » After use, allow the heating lamp to cool down to room temperature before putting it away.
- » Never leave the heating lamp unattended when in use.
- » Keep children well away from the heating lamp. A supervisor must always be present if children or handicapped persons are to be treated with the heating lamp.
- » Take care when adjusting the arm of the lamp that fingers do not get jammed in the rod.
- » The eyes should be covered with a light cloth if the heating lamp is to be used for treatment on the face to avoid the eyes becoming dry.
- » Once switched on, the lamp will become very hot and in these circumstances the heating head should on no account be touched.
- » Avoid a too rapid cooling of the body directly after a treatment session.
- » In the interests of safety always maintain a distance of at least 45 cm between the heating plate and any combustible objects or walls.
- » Avoid falling asleep during treatment sessions.
- » Never use the heating lamp if the power connecting cable is defective. Defective cables may only be repaired by authorized workshops.
- » If you are using the lamp for the first time after longer breaks it is possible that you will notice a slight smell (this has been caused by condensation). This effect is perfectly normal and will disappear after a few treatment sessions.

- » Treatment should never be painful and to avoid burns you should always ensure there is sufficient distance between the skin being treated and the heating head.
- » Please dispose of the packing material in an environmentally friendly manner.

### ***Contra-Indications***

With regard to the following contra-indications the heating lamp should not be used or only after consulting your doctor:

- » Patients with implanted electronic devices such as heart pacemakers should not treat the respective body area (for example, the chest area) with the heating lamp.
- » Do not treat those parts of the body which are numb or which are subject to sensitivity disorders, for example after the use of cold compressors.
- » Pregnant women should not treat the stomach area.
- » Women during the period two months after conception.
- » Silicone and silicone implants close to the surface on ears, nose or breast can absorb infra-red radiation and become hot. In such cases the doctor should be consulted.
- » Patients who suffer from haemophilia or other blood clotting conditions should avoid exposure to infra-red radiation because during treatment there is an increased risk of bleeding caused by vasodilatation of the blood vessels.
- » Do not use when feverish or when the body temperature is heightened.
- » In case of doubt always consult your doctor.
- » Diseased or injured skin should not be treated.
- » Fresh bleeding and haemorrhages in the treatment area should not be treated on account of the increased risk of bleeding.
- » The eyes should not be exposed to the direct heat radiation.
- » Fresh inflammation, for example on soft parts, joints and vessels in the area of application should not be treated.
- » Do not treat in cases where serious blood circulation disorders are involved in the area of application.
- » Malignant diseases in the area of application must not be treated.
- » The heating lamp must not be used to treat the area of the external sex organs.
- » Where fresh swelling in the area of application is involved.
- » Children under the age of five years should not be treated.

### Side-Effects

Treatment with the heating lamp is an agreeable type of treatment with few side-effects. However, the following side-effects can occur:

- » Reddening of the skin
- » Pronounced sweating
- » Itching of the skin
- » Increased pain

## Function Of The Heating Lamp: How it should be used

### Operating elements

1. The heating plate is located in the heating head and has been provided with a unique coating containing several minerals.
2. The arm of the lamp can be folded in and out so that the heating head can be brought into the desired position.
- 2a. Through turning the fixing elements the lamp is adjustable.
3. The required treatment duration is selected with the Timer button. Time can be set in intervals of five minutes in the range 5 – 95 minutes. However, a treatment duration of 60 minutes should not be exceeded.
4. The height of the heating lamp can be adjusted (maximum 20 cm) by retracting the base rod.
5. Power is supplied to the heating lamp via the power connecting cable.
6. The casters ensure stability on even surfaces and provide mobility in the room.

### Operating panel

7. Switch the heating lamp on and off with the Power button. The lamp is ready to operate if the switch is in the position I
8. The operations display illuminates green if the heating lamp is switched on and goes out when switched off
9. The digital display shows the following:  
After switching on with the Power button, the number of operating hours already completed  
-cont (continuous heating) after activating the Start/Stop button without a time duration input  
-the remaining treatment time, for example 15:00 means that 15 minutes of radiation remain
10. The Timer button serves to set the treatment time (adjustable from 5 min to 95 min in steps of five minutes)
11. Reset button  
-resets the time  
-is activated when the display shows "cont" to enable the setting of the treatment time
12. Start/Pause button serves to  
-start and hold the treatment if a time duration has been set  
-to start/hold the heat-up phase if a time duration has not been set

### ***Preparations for treatment***

Assemble the heating lamp for first time use. The three parts to be assembled are the treatment part, the heating head, the rod with the operating panel and the presser foot.

First remove all packaging.

Secure with the screws and nuts, the moving parts of the presser foot and place it on the floor.

### ***Assembling the heating lamp***

Insert the upper part with the operating panel (1) into the base rod (2).

Fix the upper part at the right height with the setscrew thread (3). Turn the locking screw (4) to the right.

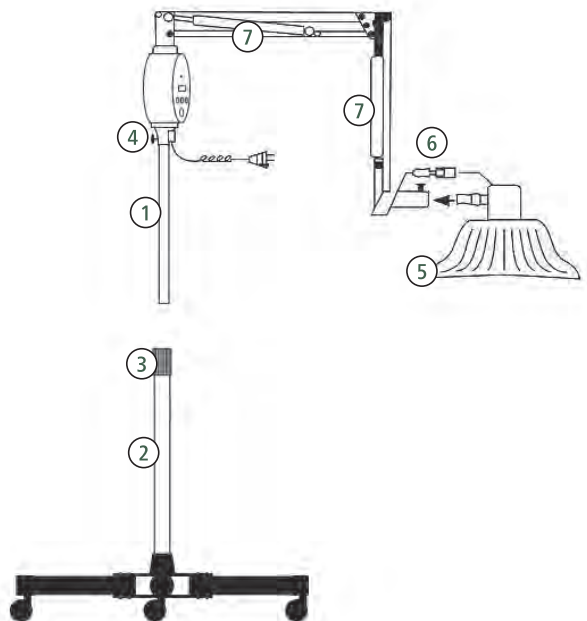
**WARNING:** Never extract the rod in excess of 20 cm.

Now insert the heating head (5), carefully turning and pressing, and connect the plug (6) on the head with the plug on the arm of the lamp.

The PierenHeat TDP Lamp is now ready to operate.

With the fixing elements (7), you can fix the set inclination of the rods.

As and when necessary carefully tighten the screws (center screw on the mineral plate or the screws on the hinge). Take care not to screw too tight to avoid damaging the screw threads.

## Realisation of the treatment

Set up the heating lamp in such a way that the rays from the heating head are directed to the required part of the body. The height is adjustable. There appears the total number of completed operating hours on the digital display. After approximately 1000 operating hours the mineral plate (and possibly the heating head) has to be replaced. Of course, when carrying out height adjustments the heating lamp should be switched off. Never carry out height adjustments if the heating head is hot.

Connect the power connecting cable plug with the power socket while taking care to ensure that the heating lamp is secure on an even surface. Switch on the lamp with the Power button.

### *Pre-heating*

There are two possibilities regarding pre-heating of the heating head:

1. With the heating lamp switched on depress the Start/Pause button. There appears in the digital display "cont", the green LED illuminates and the heating lamp heats up. The required temperature will be attained after approximately 10 – 15 minutes. The lamp does not switch off automatically in this mode.
2. Set the time with the Timer button to 10 or 15 minutes and once this time has elapsed there follows an acoustic signal and in this mode the heating lamp switches off automatically.

### *Treatment*

Now depress the Timer button on the control panel and set the required treatment duration time. Depress the Start/Pause button to start the treatment session. The green LED illuminates during treatment. Always maintain a minimum distance of 30 cm between the heating plate and the body during treatment.

A distance of approximately 30 cm. to the face should also be maintained. Treatment on the face should always be carried out in the frontal, sitting position, never in the lying position or from above so that nothing can fall onto the face or into the eyes.

When the treatment duration has been completed there follows an acoustic signal, the heating lamp switches off automatically in this mode and the green LED goes out.

### ***Recommended treatment duration***

At the beginning shorter treatment durations are recommended. This will allow you to judge whether you can tolerate the treatment or if perhaps side-effects have occurred. Observe those parts of the skin which have been treated and check for possible reactions. Immediately discontinue treatment if there are signs of over-sensitivity or allergic reactions and consult your doctor.

We would recommend one to two treatment sessions daily, each of approximately 30 to 60 minutes duration. Where acute injuries are involved treatment can be carried out over the course of several days. With chronic pain treatment can be carried out over the course of several weeks.

### ***After treatment***

Switch off the lamp with the Power button and remove the power plug from the socket. Allow the heating lamp to cool down for at least 15 minutes before putting away.

### ***Replacing the heating head***

The heating head should be replaced after 1,000 operating hours because the natural constituent parts of the heating plate have reached the end of their working life. For reasons of safety the complete heating head must be replaced, not only the heating plate. For replacement purposes use only original spare parts of the same type (CQ-BS8). Proceed as follows:

- » First ensure that the heating head has cooled right down and that the power cable has been disconnected.
- » Unscrew the plug sleeve (A) back and detach, remove plug (B) from the socket.
- » Unscrew the butterfly nuts (C) and remove the old heating head (D).
- » Insert the new heating head, tighten the butterfly nuts and re-connect the plug with the heating head socket.

### ***Heating head disposal***

At the end of its working life the heating head must be handed over to the communal recycling facility free of charge. Disposal via the household waste system is not permitted. This represents an active contribution to the protection of the environment.

## **Technical Data And General Information**

### ***Care and cleaning***

Store the lamp and accessories in a dry location at room temperature. Do not use chemical cleaning agents to clean the lamp. The heating lamp can be wiped with a damp cloth. But please first ensure that the lamp has been switched off and has completely cooled down and also that no water infiltrates the lamp.

### ***Classification***

The PierenHeat TDP heating lamp has been classified as Class IIa in compliance with Appendix IX of the Product Line 93/42/EEC in respect of medical products.

### ***Delivery content PierenHeat TDP***

Infra-Red heating lamp PierenHeat TDP	REF 306006
Operating Instructions	REF 451600-0355

### ***Delivery content PierenHeat<sup>2</sup> TDP***

Infra-Red heating lamp PierenHeat <sup>2</sup> TDP	REF 306011
Operating Instructions	REF 451600-0355

### ***Accessories***

PierenHeat TDP Infra-Red Heating Head	REF 306007
---------------------------------------	------------

### ***Warranty***

Legal right of warranty is applied according to German Civil Code.

**Technical data**

<b>PierenHeat TDP</b>	
Power supply	Alternating current, 230 V, 50 Hz
Wattage	250 W
Time setting	5 – 95 min (in 5 min steps)
Dimensions: (W x H x D)	500 x 500 x 1300 mm
Weight	ca. 10 kg
Ambient temperature when in use	5°C – 40°C
Relative humidity when in use	30% – 75%
Air pressure when in use	500 – 1060 hPa
Ambient storage temperature	-40°C – 70°C
Relative storage humidity	≤ 95%
Storage air pressure	500 – 1060 hPa
Protection class	I
Fuse specification	5 x 20 mm 3 A / 250 V
Infra-red spectrum	2 – 25 µm
Heating plate coating	Various minerals
Heating plate coating	Various minerals

The technical data apply to PierenHeat<sup>2</sup> TDP correspondingly per heating head.

## Picture symbols

Attention: Read accompanying documents, especially user manual!

Manufacturer

Date of manufacture

Serial number

Environment

Do not dispose of the appliance with the normal household waste at the end of its life, but hand it in at an official collection point for recycling. By doing this you will help to protect the environment.

**CE0482** By labelling with CE mark, the manufacturer asserts that the product fulfills all relevant requirements of the respective EC Directives. A conformation assessment process has been successfully completed. The identification number in respect of the conformation assessment process is given in accordance with the CE labelling of the relevant notified body.

Caution, hot surface

Warning; Crushing hazard: hand

Ensure that the distance between the heating head and the surface of the skin is at least 30 cm.

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Pierenkemper GmbH