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Brief instructions for use

 Before using the STIMEX® stimulation gloves, please read the instructions for use carefully and follow the pictographs on the packaging of the STIMEX® stimulation gloves. Follow the safety information for use of the product.

 Plug the connecting cable plug of the electrical stimulation device into the pushbutton adapter provided and clip this onto the respective pushbutton at the edge of the STIMEX® stimulation gloves.

 Put on both STIMEX® stimulation gloves carefully and moisten them well.

 Switch on the electrical stimulation device and increase the intensity until you clearly feel an electrical current. Perform the therapy as prescribed by your physician.

 Do not use the STIMEX® stimulation gloves on injured or diseased areas of skin.

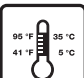
 Do not touch any metallic objects with the STIMEX® stimulation gloves during therapy.

 Do not bring the STIMEX® stimulation gloves into contact with one another during therapy.

 Do not wear any watches or jewelry, such as rings or bracelets, during therapy with the STIMEX® stimulation gloves.

 Do not touch any electronic devices, such as mobile phones or televisions, during therapy with the STIMEX® stimulation gloves.

 Please wash the STIMEX® stimulation gloves separately, at maximum 30 °C, hand wash only. Do not wring out the STIMEX® stimulation gloves!

 Do not use the STIMEX® stimulation gloves at ambient temperatures below 5 °C or above 35 °C.

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EN INSTRUCTIONS FOR USE

Intended use

STIMEX® stimulation gloves are a medical class-I product. They serve to form a full-area conductive connection between the electrical stimulation device and a patient's hands, in order to use the device as intended. The STIMEX® stimulation gloves serve for supplying electrical energy.

Indication

(Wherefore may I use the gloves?)

The gloves can be used instead of the self-adhesive electrodes and are intended for any pain at the hands. The specific syndromes are described in the instructions for use of the electro-stimulation device.

Contraindications

(When may I not use the gloves, when may I only use the gloves after consultation with the appropriate physician?)

- » Wounds and skin diseases in the foot area
- » Incompatibility with the sock materials used
- » **Caution!** Please consider additionally the contraindications of the used electro-stimulation device.

Safety information

- » Overextending the edge can cause the seam to tear on the inside insulation of the pushbutton! This can lead to direct contact with the pushbutton, which can result in damage to the skin. Please therefore make sure that the insulation is intact each time before using the product.
 - » **Do not wear any metallic jewellery**, such as brackets or piercings during therapy.
 - » Do not touch any metallic things during the therapy
 - » Do not contact the gloves with each other during the stimulation. Otherwise a short circuit occurs which would stop the stimulation. When the gloves are removed from each other again, the stimulation would continue at once. This could cause unpleasant sentiments.
 - » Always switch off the electrical stimulation device before removing the STIMEX® stimulation gloves.
 - » STIMEX® stimulation gloves can cause skin irritations due to an incompatibility or allergy to the glove materials used. If skin irritations occur, seek medical advice before continuing application.
 - » To avoid potential skin irritations, the STIMEX® stimulation gloves should only be worn as long as necessary for the therapy.
 - » The STIMEX® stimulation gloves should only be used by one person for hygienic reasons.
- ### Application
- » Plug the connecting cable plug of the electrical stimulation device into the pushbutton adapter provided and clip this onto the respective pushbutton at the edge of the STIMEX® stimulation gloves.
 - » Thoroughly soak the gloves with water. To dress more easily you can already put on the the gloves before soaking.
 - » Put on both STIMEX® stimulation gloves carefully. Follow the safety information for use of the product.
 - » Switch on the electrical stimulation device and increase the intensity until you clearly feel an electrical current. Perform the therapy as prescribed by your physician.

» Always first switch off the electrical stimulation device before removing the STIMEX® stimulation gloves.

Cleaning and care

- » Allow the STIMEX® stimulation gloves to dry in the air after therapy and after washing.
- » Please wash the STIMEX® stimulation gloves separately, at maximum 30 °C, hand wash only. If necessary, use a liquid mild detergent and then rinse the STIMEX® stimulation gloves with clear tap water.
- » Do not wring out the STIMEX® stimulation gloves!

Information on the material used for the gloves

STIMEX® stimulation gloves are made from Polyester, Elatane, Polyurethane, with interwoven silver threads.

Data on the electrical stimulation devices used/ approved

The STIMEX® stimulation gloves can be used with all electrical stimulation devices manufactured by Pierenkemper.

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