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Cleaning and care

- » Allow the STIMEX® stimulation gloves to dry in the air after therapy and after washing.
- » Please wash the STIMEX® stimulation gloves separately, at maximum 30 °C, hand wash only. If necessary, use a liquid mild detergent and then rinse the STIMEX® stimulation gloves with clear tap water.
- » Do not wring out the STIMEX® stimulation gloves!

Information on the material used for the gloves

STIMEX® stimulation gloves are made from Polyester, Elatane, Polyutethane, with interwoven stainless steel threads.

Data on the electrical stimulation devices used / approved

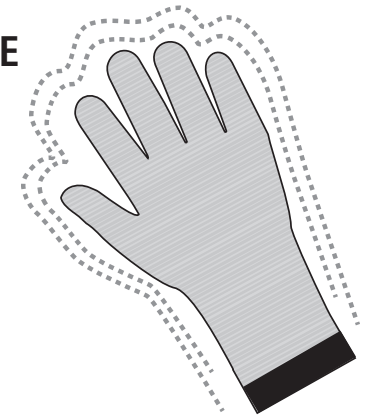
The STIMEX® stimulation gloves can be used with all electrical stimulation devices manufactured by Pierenkemper.

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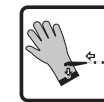
Information for use

Before using the STIMEX® stimulation gloves, please read the instructions for use carefully and follow the pictographs on the packaging of the STIMEX® stimulation gloves. Follow the safety information for use of the product.

Plug the connecting cable plug of the electrical stimulation device into the pushbutton adapter provided and clip this onto the respective pushbutton at the edge of the STIMEX® stimulation gloves.

Put on both STIMEX® stimulation gloves carefully.

Switch on the electrical stimulation device and increase the intensity until you clearly feel an electrical current. Perform the therapy as prescribed by your physician.

Do not use the STIMEX® stimulation gloves on injured or diseased areas of skin.

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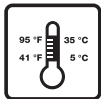
Do not touch any metallic objects with the STIMEX® stimulation gloves during therapy.

Do not bring the STIMEX® stimulation gloves into contact with one another during therapy.

Do not wear any watches or jewellery, such as rings or bracelets, during therapy with the STIMEX® stimulation gloves.

Do not touch any electronic devices, such as mobile phones or televisions, during therapy with the STIMEX® stimulation gloves.

Please wash the STIMEX® stimulation gloves separately, at maximum 30 °C, hand wash only. Do not wring out the STIMEX® stimulation gloves!

Do not use the STIMEX® stimulation gloves at ambient temperatures below 5 °C or above 35 °C.

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Intended use

STIMEX® stimulation gloves are a medical product intended for use on humans. STIMEX® stimulation gloves serve to form a full-area conductive connection between the electrical stimulation device and a patient's hands, in order to use the device as intended. The STIMEX® stimulation gloves serve for supplying electrical energy. They may only be used on uninjured skin surfaces. They are intended for application on humans. The area of application is determined by the electrical stimulation device used.

Contraindications

- » Wounds and skin diseases in the area of the gloves
- » Incompatibility with the glove materials used

Please also observe the contraindications indicated for the electrical stimulation device used.

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Safety information

- » Overextending the edge can cause the seam to tear on the inside insulation of the pushbutton! This can lead to direct contact with the pushbutton, which can result in damage to the skin. Please therefore make sure that the insulation is intact each time before using the product.
- » Do not wear any metallic jewellery, such as brackets or rings, during therapy.
- » If you do not feel enough electrical current, please moisten your hands with tap water before using the STIMEX® stimulation gloves.
- » Always switch off the electrical stimulation device before removing the STIMEX® stimulation gloves.
- » STIMEX® stimulation gloves can cause skin irritations due to an incompatibility or allergy to the glove materials used. If skin irritations occur, seek medical advice before continuing application.
- » To avoid potential skin irritations, the STIMEX® stimulation gloves should only be worn as long as necessary for the therapy.
- » The STIMEX® stimulation gloves should only be used by one person for hygienic reasons.

Application

- » Plug the connecting cable plug of the electrical stimulation device into the pushbutton adapter provided and clip this onto the respective pushbutton at the edge of the STIMEX® stimulation gloves.
- » Moisten both your hands with tap water.
- » Put on both STIMEX® stimulation gloves carefully. Follow the safety information for use of the product.
- » Switch on the electrical stimulation device and increase the intensity until you clearly feel an electrical current. Perform the therapy as prescribed by your physician.
- » Always first switch off the electrical stimulation device before removing the STIMEX® stimulation gloves.