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**Brief Instructions for Use**

Before using the STIMEX® stimulation socks, please read the instructions for use carefully and follow the pictographs on the packaging of the STIMEX® stimulation socks. Follow the safety information for use of the product.

Plug the connecting cable plug of the electrical stimulation device into the pushbutton adapter provided and clip this onto the respective pushbutton at the edge of the STIMEX® stimulation socks.

Put on both STIMEX® stimulation socks carefully.

Switch on the electrical stimulation device and increase the intensity until you clearly feel an electrical current. Perform the therapy as prescribed by your physician.

Do not use the STIMEX® stimulation socks on injured or diseased areas of skin.

Do not touch any metallic objects with the STIMEX® stimulation socks during therapy.

Do not bring the STIMEX® stimulation socks into contact with one another during therapy.

Do not wear any metallic jewellery, such as anklets or toe rings, during therapy.

Please wash the STIMEX® stimulation socks separately, at maximum 30 °C, hand wash only. Do not wring out the STIMEX® stimulation socks!

Do not use the STIMEX® stimulation socks at ambient temperatures below 5 °C or above 35 °C

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## EN INSTRUCTIONS FOR USE

### Intended use

STIMEX® stimulation socks are a medical class-I product. They serve to form a full-area conductive connection between the electrical stimulation device and a patient's feet, in order to use the device as intended. The STIMEX® stimulation socks serve for supplying electrical energy.

### Indication (Wherefore may I use the socks?)

The socks can be used instead of the self-adhesive electrodes and are intended for any pain at the feet. The specific syndromes are described in the instructions for use of the electro-stimulation device."

### Contraindications

» Wounds and skin diseases in the foot area  
» Incompatibility with the sock materials used

» **Caution!** Please consider **additionally** the contraindications of the used electro-stimulation device.

### Safety information

» Overextending the edge can cause the seam to tear on the inside insulation of the pushbutton! This can lead to direct contact with the pushbutton, which can result in damage to the skin. Please therefore make sure that the insulation is intact each time before using the product.

» **Do not wear any metallic jewellery**, such as anklets or toe rings, during therapy.

» **Do not touch any metallic things** during the therapy.

» Do not contact the socks with each other during the stimulation. Otherwise a short circuit occurs which would stop the stimulation. When the socks are removed from each other again, the stimulation would continue at once. This could cause unpleasant sentiments.

» STIMEX® stimulation socks can cause skin irritations due to an incompatibility or allergy to the sock materials used. If skin irritations occur, seek medical advice before continuing application.

» To avoid potential skin irritations, the STIMEX® stimulation socks should only be worn as long as necessary for the therapy.

» The STIMEX® stimulation socks should only be used by one person for hygienic reasons.

### Application

» Plug the connecting cable plug of the electrical stimulation device into the pushbutton adapter provided and clip this onto the respective pushbutton at the edge of the STIMEX® stimulation socks.

» You can use just 1 sock or both at the same time. You can also use 1 sock with 1 self-adhesive electrode. For this issue take a look in the recommendations of the electrode placement in the instructions for use of the electro-stimulations device. The sock replaces the self-adhesive electrode at the foot in this recommendation.

» Thoroughly soak the socks with water. To dress more easily you can already put on the socks before soaking.

» Put on both STIMEX® stimulation socks carefully. .

» Switch on the electrical stimulation device and increase the intensity until you clearly feel an electrical current. Perform the therapy as prescribed by your physician.

» Always first switch off the electrical stimulation device before removing the STIMEX® stimulation socks.

### Cleaning and care

» Allow the STIMEX® stimulation socks to dry in the air after therapy and after washing.

» Please wash the STIMEX® stimulation socks separately, at maximum 30 °C, hand wash

only. If necessary, use a liquid mild detergent and then rinse the STIMEX® stimulation socks with clear tap water.

» Do not wring out the STIMEX® stimulation socks!

### Information on the material used for the socks

STIMEX® stimulation socks are made from Polyester, Elatane, Polyutethane, with interwoven silver threads.

### Data on the electrical stimulation devices used / approved

The STIMEX® stimulation socks can be used with all electrical stimulation devices manufactured by Pierenkemper.

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