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Cleaning and care

- » Allow the STIMEX® stimulation socks to dry in the air after therapy and after washing.
- » Please wash the STIMEX® stimulation socks separately, at maximum 30 °C, hand wash only. If necessary, use a liquid mild detergent and then rinse the STIMEX® stimulation socks with clear tap water.
- » Do not wring out the STIMEX® stimulation socks!

Material

STIMEX® stimulation socks are made from Polyester, Elastane, Polyurethane, with interwoven stainless steel threads.

Data on the electrical stimulation devices used / approved

The STIMEX® stimulation socks can be used with all electrical stimulation devices manufactured by Pierenkemper.

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Information for use

Before using the STIMEX® stimulation socks, please read the instructions for use carefully and follow the pictographs on the packaging of the STIMEX® stimulation socks. Follow the safety information for use of the product.

Plug the connecting cable plug of the electrical stimulation device into the pushbutton adapter provided and clip this onto the respective pushbutton at the edge of the STIMEX® stimulation socks.

Put on both STIMEX® stimulation socks carefully.

Switch on the electrical stimulation device and increase the intensity until you clearly feel an electrical current. Perform the therapy as prescribed by your physician.

Do not use the STIMEX® stimulation socks on injured or diseased areas of skin.

Pierenkemper GmbH
Am Geiersberg 6 · 35630 Ehringshausen · Deutschland
Tel. +49 6443 8333-175 · Fax +49 6443 8333-169
info@pierenkemper.eu · www.pierenkemper.eu

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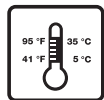
Do not touch any metallic objects with the STIMEX® stimulation socks during therapy.

Do not bring the STIMEX® stimulation socks into contact with one another during therapy.

Do not wear any metallic jewellery, such as anklets or toe rings, during therapy.

Please wash the STIMEX® stimulation socks separately, at maximum 30 °C, hand wash only. Do not wring out the STIMEX® stimulation socks!

Do not use the STIMEX® stimulation socks at ambient temperatures below 5 °C or above 35 °C

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Intended use

STIMEX® stimulation socks are a medical product intended for use on humans. STIMEX® stimulation socks serve to form a full-area conductive connection between the electrical stimulation device and a patient's feet, in order to use the device as intended. The STIMEX® stimulation socks serve for supplying electrical energy. They may only be used on uninjured skin surfaces. They are intended for application on humans. The area of application is determined by the electrical stimulation device used.

Contraindications

- » Wounds and skin diseases in the foot area
- » Incompatibility with the sock materials used

Please also observe the contraindications indicated for the electrical stimulation device used.

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Safety information

- » Overextending the edge can cause the seam to tear on the inside insulation of the pushbutton! This can lead to direct contact with the pushbutton, which can result in damage to the skin. Please therefore make sure that the insulation is intact each time before using the product.
- » Do not wear any metallic jewellery, such as anklets or toe rings, during therapy.
- » If you do not feel enough electrical current, please moisten your feet with tap water before using the STIMEX® stimulation socks.
- » Always switch off the electrical stimulation device before removing the STIMEX® stimulation socks.
- » STIMEX® stimulation socks can cause skin irritations due to an incompatibility or allergy to the sock materials used. If skin irritations occur, seek medical advice before continuing application.
- » To avoid potential skin irritations, the STIMEX® stimulation socks should only be worn as long as necessary for the therapy.
- » The STIMEX® stimulation socks should only be used by one person for hygienic reasons.

Application

- » Plug the connecting cable plug of the electrical stimulation device into the pushbutton adapter provided and clip this onto the respective pushbutton at the edge of the STIMEX® stimulation socks.
- » Moisten both your feet with tap water.
- » Put on both STIMEX® stimulation socks carefully. Follow the safety information for use of the product.
- » Switch on the electrical stimulation device and increase the intensity until you clearly feel an electrical current. Perform the therapy as prescribed by your physician.
- » Always first switch off the electrical stimulation device before removing the STIMEX® stimulation socks.