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Addition for the operating manual of PierenHeat

Additional safety instructions for the use of the device:

- **Danger/Caution:** To avoid the risk of electric shock, this device may only be connected to a mains supply with a protective earth conductor.
- Never use the product if it functions incorrectly or has been damaged. If, contrary to expectations, malfunctions occur, please contact our service team.
- **Warning:** You shouldn't use this device next to or stacked with other devices, as this may result in incorrect operation.
- **Warning:** Keep a distance of at least 30 cm (12 inches) between the PierenHeat and a high frequency communication device (e.g. mobil phone), including its accessories.
- The product may only be used with original accessories.
- Device mains plugs and/or socket outlet must be easily accessible during operation of the device, so that disconnection from the mains supply is possible quickly if necessary.
- Only use at temperatures between 5 °C and 40 °C, at a relative humidity between 30 % and 75 % and a pressure between 50 kPA and 106 kPA. Do not use the product in bathroom or similar humid environments
Warning: Do not operate the device in the vicinity of explosive and/or flammable substances or steam!
- In case of commercial use of the product in Germany the manufacturer recommends a technical check on the product every 2 years. This check is an obligation by §11 MPBetreibV. Please observe the applicable legal regulations of your country.